


Break Apart Probe

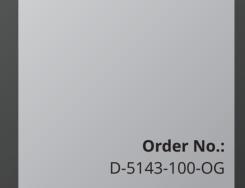


Description

XL PAX5 BA consists of an orange-labeled probe hybridizing to the PAX5 gene and distal gene regions at 9p13 and a green-labeled probe hybridizing proximal to the PAX5 gene region at 9p13.

Clinical Details

PAX5 (paired box 5), also known as B-cell specific activator protein (BSAP), is located on 9p13 and is a member of the paired box (PAX) family of transcription factors. The central characteristic is a highly conserved DNA-binding motif (paired box). PAX5 is a master regulator of B cell identity and development. It is expressed from early pro-B stage until final plasmacytic differentiation playing a dual role by activating B-cell commitment genes while repressing non-B-lineage genes. PAX5 alterations in B-cell acute lymphoblastic leukemia (B-ALL) include deletions of the whole gene, focal deletions, sequence mutations, intragenic amplification, and rearrangements. PAX5 rearrangements occurred at an incidence of about 2.5% in childhood B-ALL (Nebral et al) and PAX5 internal rearrangements are observed in 21% of B-ALL patients harboring 9p abnormalities (Coyaud et al). Two B-ALL subtypes, PAX5P80R and PAX5-altered (PAX5alt) are defined by different gene expression profiles and differential PAX5 alterations and considered as potential novel entities in the 5th edition of the World Health Organization (WHO) Classification of Haematolymphoid Tumours (2022). PAX5alt is characterized by various PAX5 alterations including chromosomal rearrangements resulting in dysregulated expression of PAX5 and subsequent transcriptional targets. Until 2019, 24 fusion partner genes of PAX5 were found, the most frequent being ETV6, NOL4L, AUTS2, CBFA2T3. PAX5::ETV6 results from dic(9;12)(p13;p13) which is associated with early suppression of B-cell differentiation. In PAX5::IGH/ t(9;14)(p13;q32), identified in a subset of aggressive B-cell non-Hodgkin lymphoma, the intronic Eµ enhancer of the IGH locus is juxtaposed next to the upstream promoter of PAX5, resulting in increased PAX5 expression.

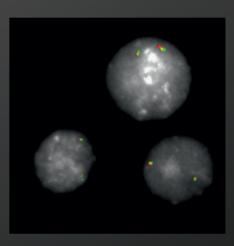

XL PAX5 BA hybridized to lymphocytes. Three normal interphases are shown.

Clinical Applications


■ ALL

Literature

- Cobaleda et al (2007) Nat Immunol 8:463-470
- Nebral et al (2009) Leukemia 23:134-143
- Coyaud et al (2010) Blood 115:3089–3097
- Gu et al (2019) Nat Genet 51:296-307

FACTSHEET



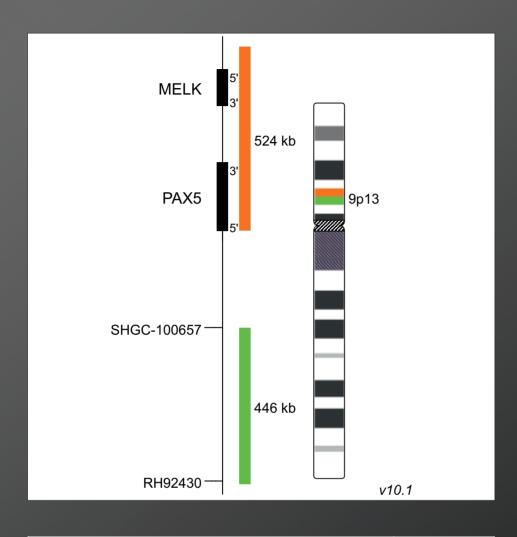

Aberrant Cell (typical results): One green-orange colocalization/fusion signal (1GO), one separate green (1G) and orange (1O) signal each resulting from a chromosome break in the respective locus.

MetaSystems Probes

EUROPE & RUSSIA

Germany, Altlussheim info@metasystems-probes.com

Italy, Milano info@metasystems-italy.com

AMERICAS

USA, Medford info@metasystems.org

Argentina, Buenos Aires info@metasystems-latam.com

ASIA & INDIA

China, Hong Kong info@metasystems-asia.com

China, Taizhou info@metasystems-china.com

India, Bangalore info@metasystems-india.com

FACTSHEET



